<b>IDEAGRAFICA S.r.l.</b>				IMPIANTO DI PROPRIETÀ/OWNER <b>BIOFARMA</b>			
PRODOTTO/PRODUCT <b>Esox Israele</b>		CODICE/CODE <b>12014109165205001</b>		IDG FUST.	TRACCIATO/ DIE CUT NUMBER	DIMENSIONI/SIZE <b>115x504</b>	DIMENSIONE PIEG./ FOLDED SIZE <b>115x42</b>
N. COLORI/ No OF COLORS <b>1/1</b>	COLOR 1 <b>NERO</b>	COLOR 2	COLOR 3	COLOR 4	COLOR 5	COLOR 6	COLOR 7
DATA/DATE <b>31/7/23</b>	IDG Nr. <b>Bozza 1</b>	OPERATOR <b>LUCA</b>	TESTO BRAILLE/BRAILLE TEXT		LAETUS	CARTONE/CARTON TYPE <b>UM 50 gr</b>	

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## Ezox

Medical device intended to treat gastro-oesophageal reflux  
Each stick pack contains a 10 ml single dose of ready-to-use oral solution.

**INDICATIONS: EZOX** is a medical device with a mechanical action that allows rapid relief from the symptoms of gastro-oesophageal reflux: pyrosis, epigastric pain, acid regurgitation, irritating cough and dysphonia.

Gastro-oesophageal Reflux Disease (GERD) is characterised by symptoms (pyrosis, acid reflux, irritative cough and dysphonia) with or without concomitant erosion of the oesophageal mucosa.

GERD is usually exacerbated when lying down and symptoms can therefore be worse at night time.

**EZOX** is an original and innovative hyaluronic acid and chondroitin sulphate complex, that protects the oesophageal mucous membrane.

The inclusion in the formulation of **EZOX** of a highly bioadhesive component (poloxamer 407), allows the hyaluronic acid and

chondroitin sulphate to adhere to the wall of the oesophagus and therefore provides better protection against the hydrochloric acid produced by the stomach.

Chondroitin sulphate also favours the oesophageal mucosal repair process.

### COMPOSITION:

Hyaluronic acid, chondroitin sulphate, polyvinylpyrrolidone, xylitol, sodium benzoate, potassium sorbate, flavourings, purified water, poloxamer 407.

### PACK:

Each package contains 20 stick packs. One stick pack contains a 10 ml single dose of ready-to-use oral solution.

**DOSE AND INSTRUCTIONS FOR USE:** One 10 ml stick pack of ready-to-use oral solution for drinking after main meals and at bedtime, or as instructed by your doctor.

### CONTRAINDICATIONS:

There are no specific contraindications, except in cases of hypersensitivity to any of the components in the product. In these cases, stop the treatment and contact your doctor.

Although there are no known side effects, it is advisable not to use the product during pregnancy or during the first months of breastfeeding.

In any case, it is advisable to consult your doctor before use. There are no known interactions with other oral products.

**WARNINGS:** Do not exceed the recommended dose. Contact your doctor if symptoms persist or side effects occur. Do not use the product after the expiration date stated on the package. Store the product in a cool, dry place, away from light and heat sources. Do not freeze the product. Keep the product out of reach of children. Do not administer the product to children under the age of 12.

**Manufacturer:** Biofarma  
Via Castelliere, 2 - 33036 Mereto di Tomba (UD) – Italy

**Registration holder:**  
Megapharm Ltd., 15 Ha'tidhar st., Ra'anana, Israel, Telephone:  
09-7604596

Ministry of Health medical device license no. 27570007

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